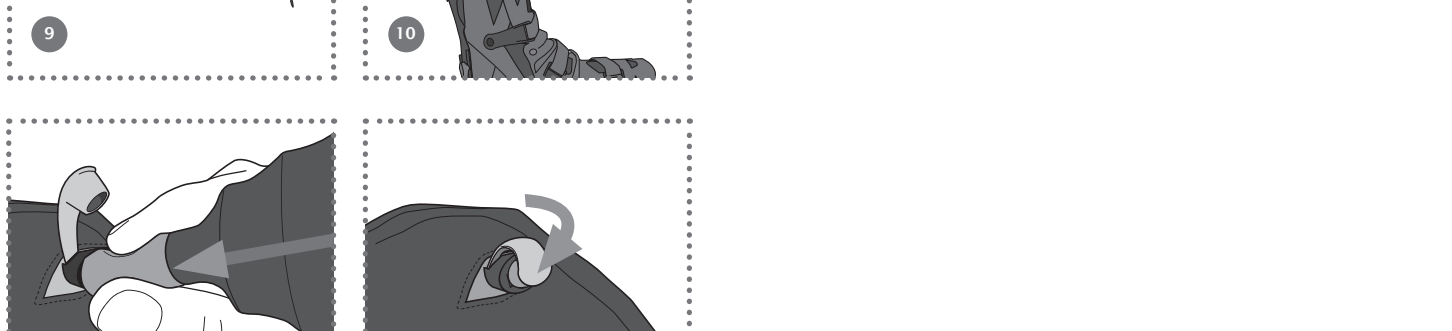
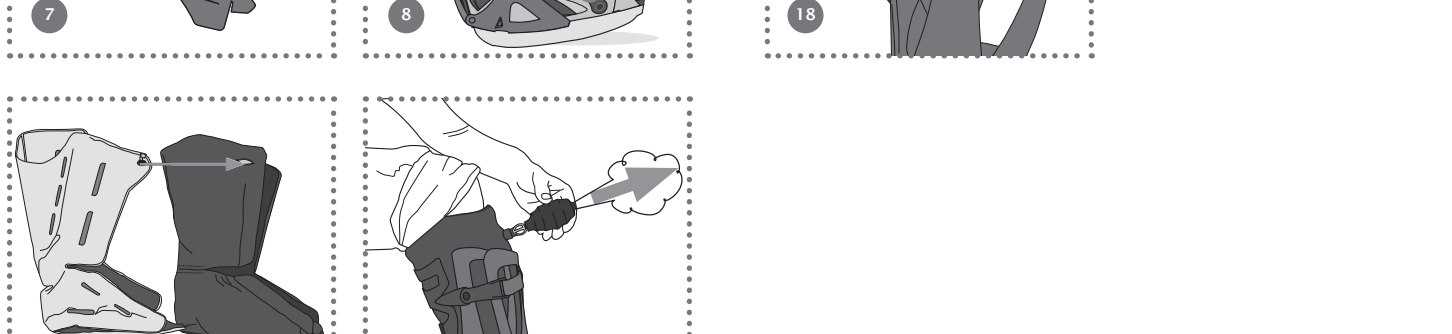
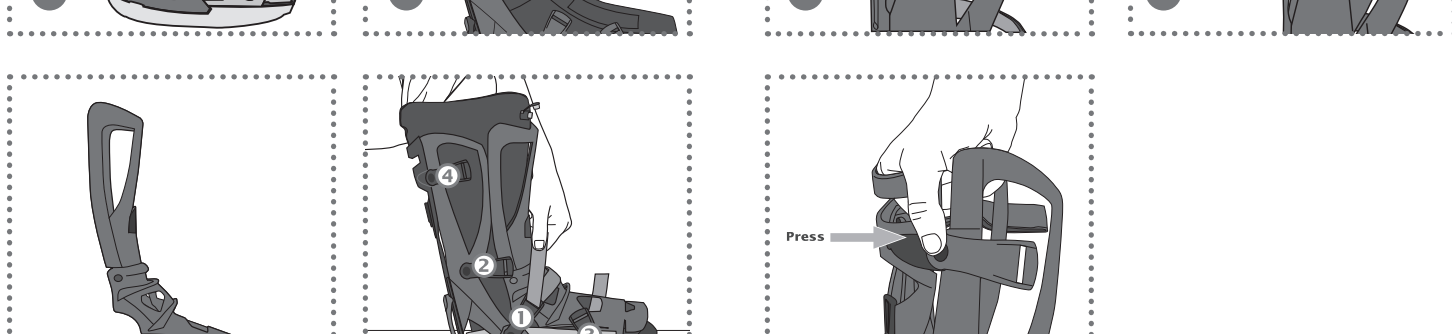
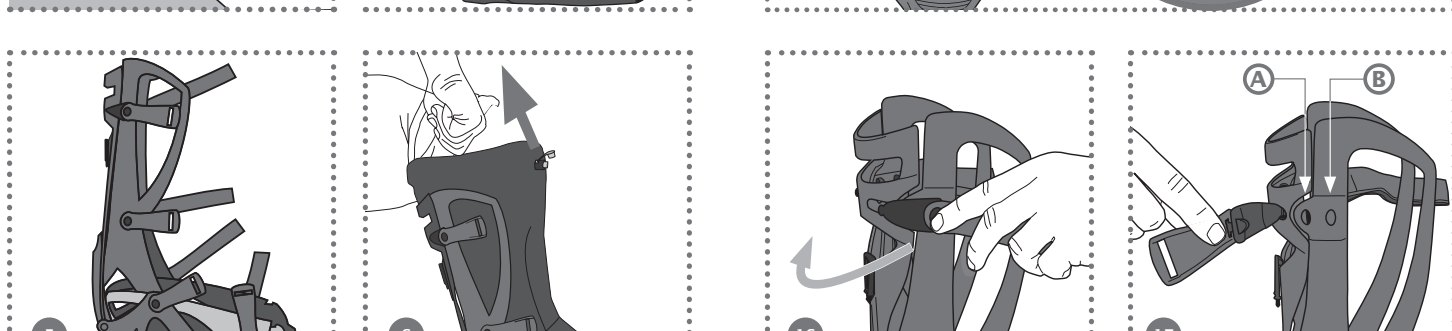
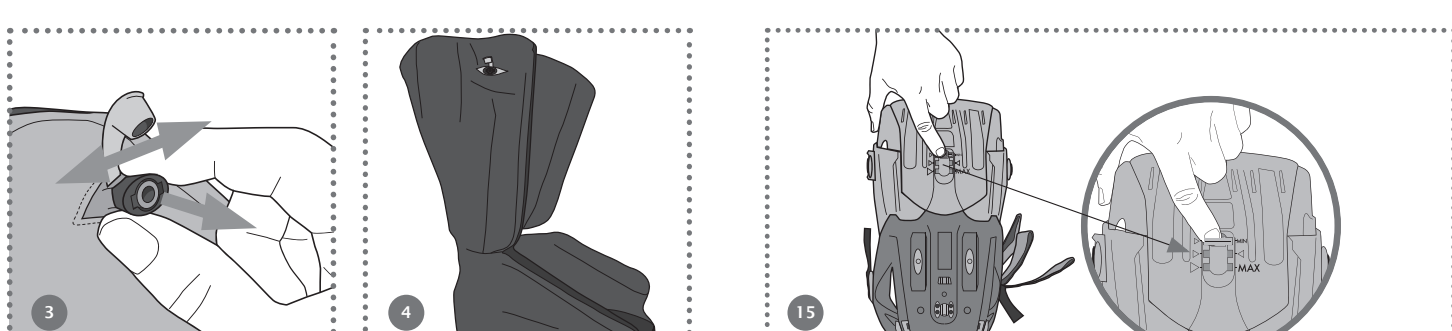
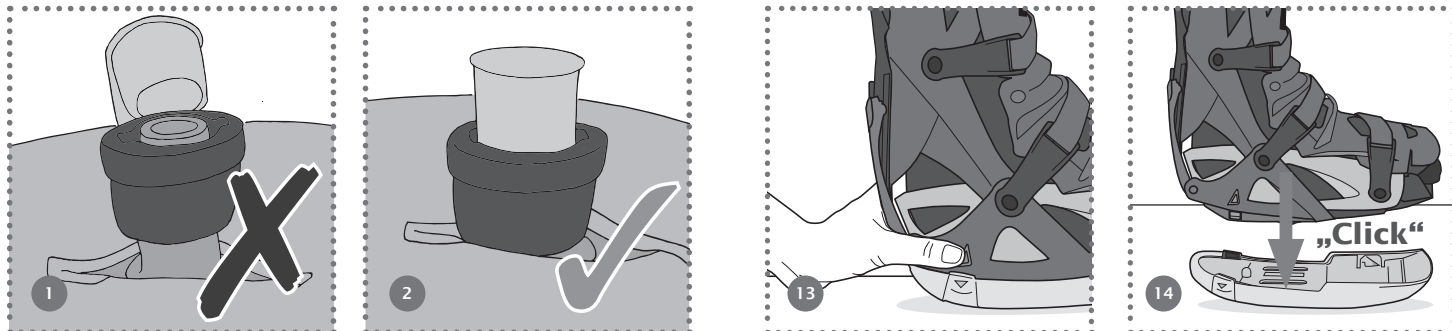
English

Purpose: Lower leg and foot orthosis for relief and immobilisation in defined adjustable positions.

Safety information

△ The orthosis should only be fitted and its use explained by trained personnel.

△ The product is intended only for use on a single patient.

△ VACOPed Diabetic is not designed for dynamic stresses such as jumping and fast walking. If you fall with the VACOPed Diabetic fitted, check your VACOPed Diabetic immediately for damage. If the VACOPed Diabetic is damaged, it can impair your safety.

△ Never wear the product over open wounds and only wear it after prior medical instruction.

△ The patient may only change the position of the device after consultation with the physician responsible for their care.

△ If excessive pain, numbness or sensation changes occur, or if unusual reactions or an unpleasant feeling are experienced while wearing the device, please contact your doctor or orthopaedic technician without delay.

△ This orthosis has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other activities.

△ When used in combination with another medical product, consult your doctor.

△ Only spare and/or replacement parts approved by OPED GmbH should be used.

△ Please note the Road Traffic Act applicable in your country.

△ Note: Not made from natural rubber latex.

△ Check that the sole fits securely, otherwise there is a risk of slipping.
Sole: When the toe section is fully extended, the gait may be modified or restricted.

△ The small spheres in the vacuum cushion can slide around. We therefore recommend opening the valve every two to three days, smoothing any collections of small spheres out and then restoring the vacuum.

△ Storage: The valve flap must always be closed.
The valve ring must be pushed downwards and the valve flap closed (Figure 1 + 2).

△ This product is intended to be used for no more than 6 months.

How to use

Fitting the VACOPed Diabetic

When fitting the VACOPed Diabetic, it can be found in "valve functions" to allow together, pull the valve ring upwards (Figure 1) and pull the valve ring downwards (Figure 2). If the small spheres in the liner (Figure 4) are grouped together, they will move smoothly. Place your foot in the fabric liner (Figure 4) and position it with the fabric liner all the way back in the lower shell (Figure 3). Then close the fabric liner by pulling the lower strap upwards (Figure 3). When fitting the pump, open the valve towards the lower shell; bottom tabs into the lower shell and close the strap by hand and by the heel lever (see Section 10) and the footrest remains flat and smoothens the footrest (see illustration) (Figure 3). The frame holder for the sole must be positioned so that the sole is not damaged.

△ Adjusting and securing the vacuum cushion.
Readjustable vacuum cushion: Inside the liner, there is a cushion containing thousands of small spheres, in which adapts perfectly to the anatomy and any swelling of your ankle joint. Pressure-sensitive areas and swellings can be relieved through the targeted redistribution of these small spheres. To ensure that the spheres do not slide around inside the cushion, the air is extracted via the valve using a suction pump, creating a vacuum (Figure 12).

△ Closing the valve.
Pull the valve ring downwards and close the valve cap. This sets the vacuum (Figure 12).

△ Valve functions.
Note: The replacement liner does not contain a cushion! When changing the liner, the cushion must be removed from the used liner and inserted into the new one! Opens the rear zip fastener all the way and remove the vacuum cushion from the used liner. Open the valve and shake out the cushion so that the small spheres can redistribute themselves (see Valve functions). Establish the vacuum (see Valve functions). Place the vacuum cushion in the new liner and slide the valve through the bottom hole (Figure 12).

△ Changing the cushion.
Note: The replacement liner does not contain a cushion! When changing the liner, the cushion must be removed from the used liner and inserted into the new one! Opens the rear zip fastener all the way and remove the vacuum cushion from the used liner. Open the valve and shake out the cushion so that the small spheres can redistribute themselves (see Valve functions). Establish the vacuum (see Valve functions). Place the vacuum cushion in the new liner and slide the valve through the bottom hole (Figure 12).

△ Fitting and changing the sole.
Push both unhooking buttons simultaneously (Figure 13).

△ Removing the sole.
Push both unhooking buttons simultaneously (Figure 13).

△ Fitting the sole.
Position the VACOPed Diabetic straight and centred on the sole before pushing it against the sole (you will hear two clicks) (Figure 14).

△ Adjusting the sole length.
Remove the sole (see Fitting the sole) and push the unhooking button. Pull the toe section to the desired length (Figure 15).

△ Adjusting the calf circumference.
Push the buckle out of the lower shell (Figure 16). Push the buckle to the desired position. Position A = large calf circumference; Position B = small calf circumference (Figure 17). Push the buckle into the required hole (Figure 18).

△ Repeat the process on the opposite side.

Repeat information

Pressure points / problem-solving.
Pull the valve ring to allow air to flow into the cushion. Spread the cushion filling evenly at the pressure point. Re-fit the orthosis (see how to use). Using a finger, check that there are enough small spheres in the cushion between the pressure point and the outer lower shell.

Showering / washing.
You can shower / enter water with the product if your doctor has given you permission to do so.

Sleeping.
Before you go to sleep, place a towel, such as a hand towel, underneath the device so that the bed sheets are not damaged.

Product characteristics

Sizes.
The product can be worn on the left or right.
Recommendation: At the size interfaces between the Small/Medium and Medium/Large versions, supply the smaller VACOPed Diabetic size if possible.

	Small	Medium	Large
EUR	35 - 39	39 - 43	43 - 49
UK	2 - 5	4.5 - 9	8.5 - 15
US (M)	35 - 39	39 - 43	43 - 49
US (M)	2.5 - 5.5	5 - 9.5	9 - 15.5
US (F)	4.5 - 8	7.5 - 12	11.5 - 14

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