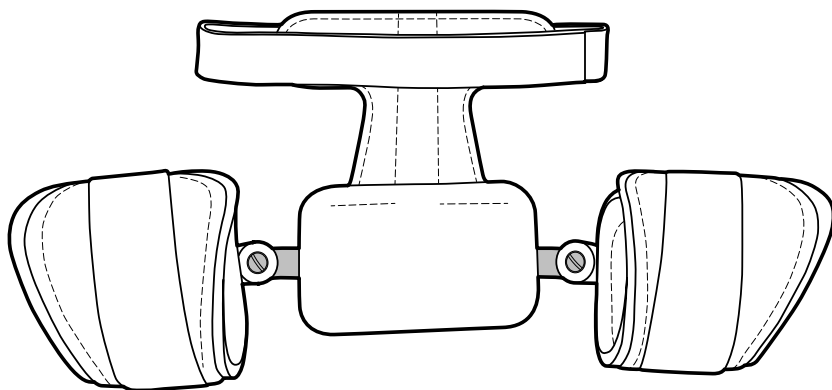
# Model 2990

Dynamic Hip Abduction Orthosis

Technische gebruiksaanwijzing  
Technische Gebrauchsanleitung  
Technical manual  
Manuel technique  
Teknisk instruktion  
Teknisk manual  
Tekniset ohjeet

**Basko**  
Healthcare

[basko.com](http://basko.com)

Technische gebruiksaanwijzing .....	4
Technische Gebrauchsanleitung .....	6
Technical manual .....	8
Manuel technique .....	10
Teknisk instruktion .....	12
Teknisk instruktion .....	14
Teknisk manual.....	16
Tekniset ohjeet .....	18

NL

DE

ENG

FR

SE

DK

NO

FI

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

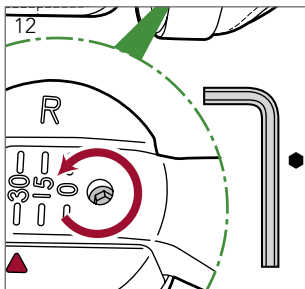
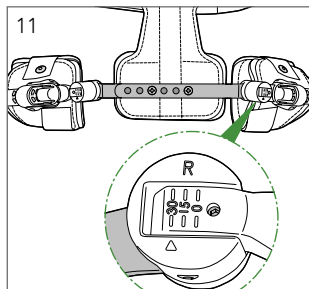
# Construction Hip Abduction Orthosis

## Fitting and adjusting the orthosis
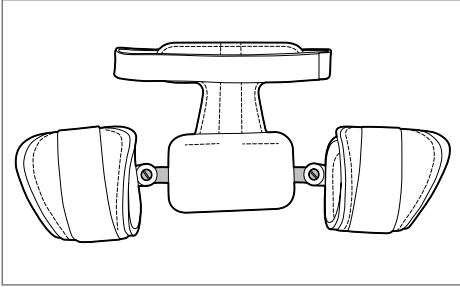
### Front 2990




### Back 2990


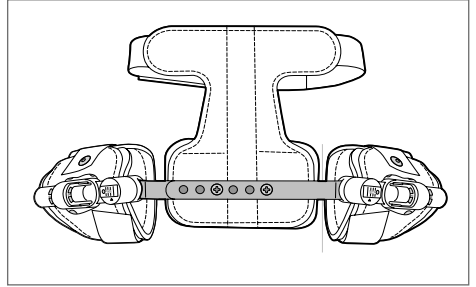
### Step 1

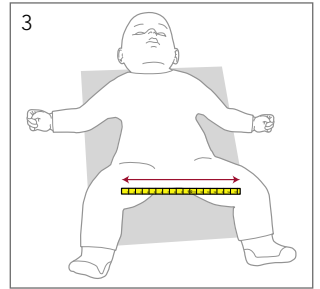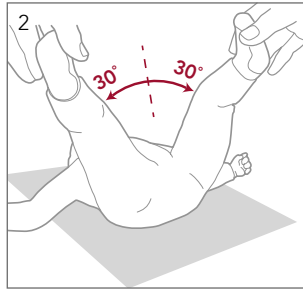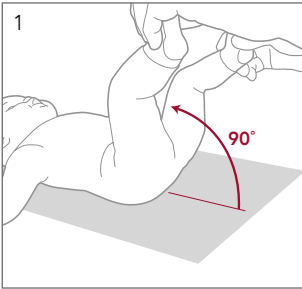Lay the child on it's back and position the hips in 90° flexion (Fig. 1) and the legs each in 30° abduction (Fig. 2). Measure the distance between both knees (Fig. 3).

### Step 2


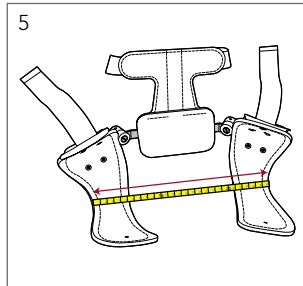
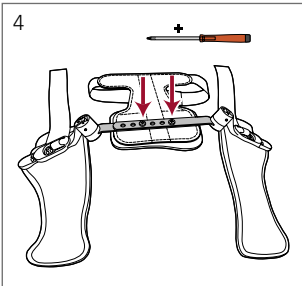
The width of the metal frame can be adjusted by removing both Phillips head screws at the back of the orthosis (Fig.4). Optionally, the Allen key provided with the orthosis can be used on the flange nuts at the inside of the frame for easier removal of the screws. Set the width of the frame so the distance between the cuffs match as close as possible the distance measured in step 1 (Fig. 5). Reposition the screws in the desired position and tighten them securely into the flange nuts.



### Step 3

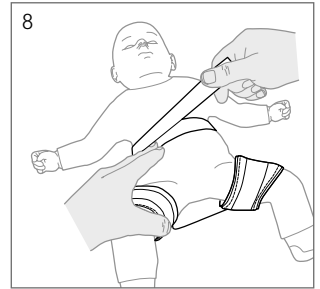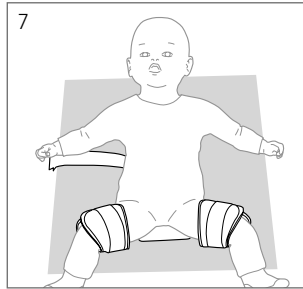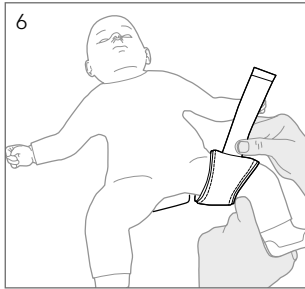Position the orthosis flat down after opening the Velcro closures of the chest strap and both thigh cuffs. Lay the child down with it's buttocks on the back part. Shape the cuff around the leg and the transparent flap (Fig. 6) and close the Velcro. Repeat with the other leg (Fig. 7) and close the chest strap (Fig. 8).



### Step 4

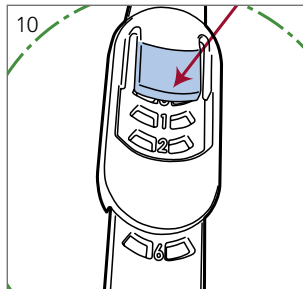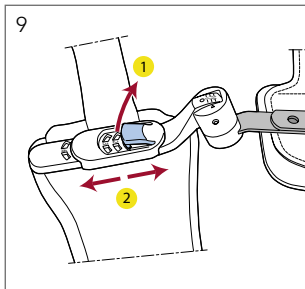The play between the cuff and the popliteal area can be up one finger thickness max. If necessary, the cuff can be slid slightly forward or backward for fine tuning. Unlock the tab from the lock by pushing it up (Fig. 9.1) and slide the cuff into the optimum position (Fig. 9.1). Release the tab until it clicks and locks again. For correct symmetry of the cuffs the slits are numbered (Fig. 10).

### Step 5

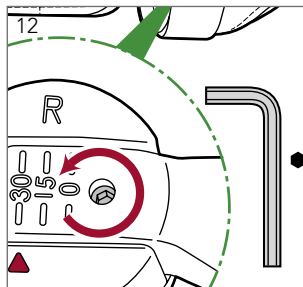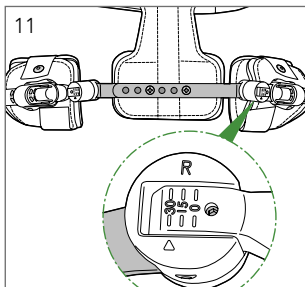By default, the dynamic joint of the orthosis is set in 0° to 90° Flexion and 30° to 0° Abduction (Fig. 11). Only when indicated by the physician, the abduction angle can be infinitely adjusted to 0°. Insert the supplied Allen key into the small opening next to the protractor, at the back of the joint. Turn it clockwise to reduce the abduction (Fig. 12). The triangle below the protractor is your point of reference.

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