allard

Support for better life!

Instructions for Use

Basic Wrist Support

Short

Item No. 27441

SV BASIC Wrist Support Short

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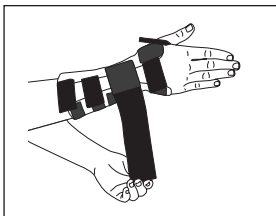
Indications / Intended Use

Basic Wrist Support Short is intended for instability in the wrist in combination inflammations, work overload, joint stress and rheumatic problems. Single Patient-multiple use.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from: 74% Cotton, 19% Nylon and 7% Lycra with an aluminium support. Shape the aluminium support carefully to fit the shape of the thumb. Avoid repetitive bending. This product does not contain Latex, and is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

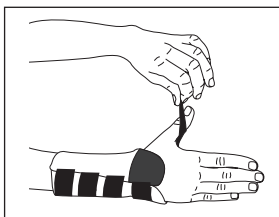
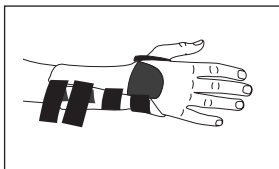
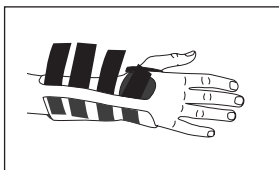
Disposal in accordance with local legislation.

Information


Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

**Instruction for use**

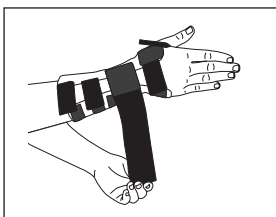1. Check that the support is complete and without damage.
2. Open all Velcro straps.
3. Place the hand over the support with the inside facing the aluminum spoon.
4. Secure all Velcro straps. Adjust the thumb strap.
5. For the long model, secure the extra circular strap around the wrist. Adjust for comfort.

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