GRIPA 28210

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Indications / Intended Use

This product is intended to give a light support for the wrist and to help prevent work related injuries - most commonly with repetitive work patterns. The material in the palm enhances the gripping ability.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This orthosis is made from polyester, cotton and elastane with a filling made of polyurethane. The anti-slip material in the palm is made from synthetic rubber. This product is not flame retardant. Latex free.


Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.


Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instruction for use

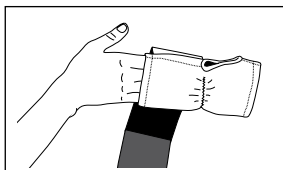
1. Check that the support is complete and without damage.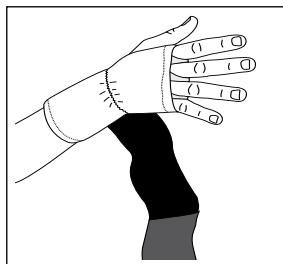

2. Put the hand into the support with the rubber material facing the outside on the palm side.

3. Make sure that the finger divider straps are correctly positioned.

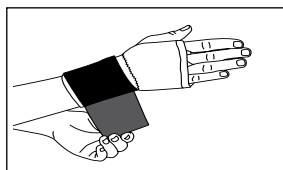4. Pull the wide elastic strap around the wrist and secure.

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