allard

Support for better life!

Instructions for Use

Selection[®]
Childrens wrist
Art. No. 35305

Instructions For Use

EN	SELECTION® CHILDRENS WRIST	p. 4
SV	SELECTION® CHILDRENS WRIST	p. 5
DA	SELECTION® CHILDRENS WRIST	p. 6
NO	SELECTION® CHILDRENS WRIST	p. 7
FI	SELECTION® CHILDRENS WRIST	p. 8
IT	SELECTION® CHILDRENS WRIST	p. 9
NL	SELECTION® CHILDRENS WRIST	p. 10
DE	SELECTION® CHILDRENS WRIST	p. 11
FR	SELECTION® CHILDRENS WRIST	p. 12
ES	SELECTION® CHILDRENS WRIST	p. 13

35305 SELECTION CHILDRENS WRIST

Carefully read this instruction sheet before use!

Intended use

To guide the hand and wrist to a functional position. To alleviate pain in inflammation and rheumatic disorders. To reduce spasticity in neurological conditions in the hand and wrist.

Indications

Rheumatic disorders. Neurological disorders where wrist position need mild correction or support. After casting. Inflammatory conditions. Single Patient-multiple use.

Contraindications

No known.

Adjustments

Fitter should make sure the stay is properly adjusted. This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from polyester, nylon and elasthan. Spoons are made from aluminum. This product is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Remove stay before laundering. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal


Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

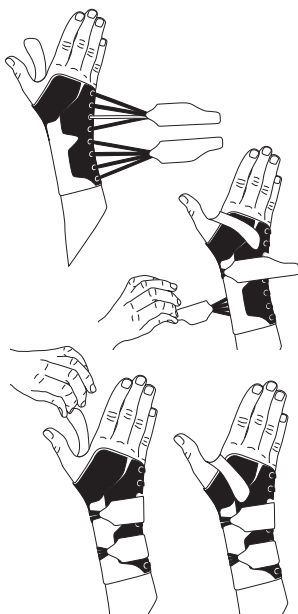**Instructions for use**

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the two straps around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

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© Camp Scandinavia_0226_Rev. 02