Gripper™3

Hinged Knee

Instructions For Use

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

EC

REP

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Gripper is a trademark of Medical Specialties, Inc.
Med Spec is a registered trademark of Medical Specialties, Inc.
1/25

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A. Disengage all closure straps. Position knee brace so opening closes on the anterior (front) side of the leg and the patella cut-out is centered around the patella (kneecap). Condylar pads may be repositioned medially or laterally by a medical professional to provide a custom fit.

B. Fasten inner (wide) flaps above and below the knee. Make sure patella is centered on patella cut-out and adjust position of knee brace if necessary.

C. Fasten outer straps above and below the knee. If an outer strap is too long, your medical professional may remove the Y shaped “alligator” hook fastener tab from the strap, cut the strap in small increments to the desired length, and then reattach the “alligator” fastener to the end of the strap.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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