
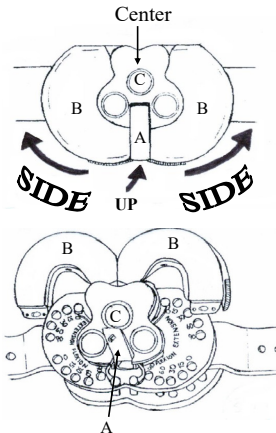
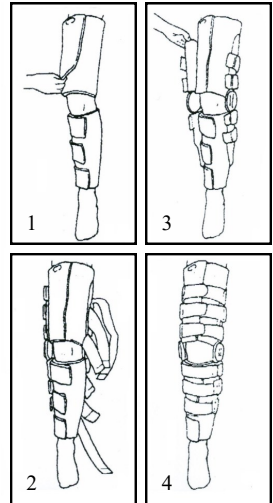
POST-OP R.O.M. KNEE BRACE

25082—25084 SERIES

25092—25094 SERIES

25182—25184 SERIES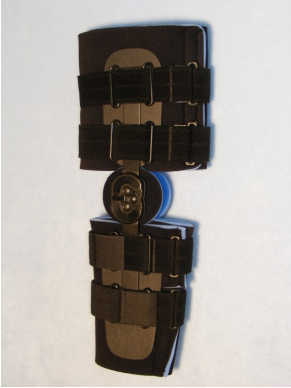

EC REP

Code of Practice AB
Brunnsgatan 48, 553 13 Jönköping, Sweden

Made in Mexico

APPLICATION

1. Open straps. Snuggly wrap foam liners around thigh and calf with closures in front.
2. Position lateral hinge with center of hinge (C) in alignment with top edge of knee cap and in midline of leg. Use hook on plastic paddle to secure to liner.
3. Repeat to secure medial hinge. Make sure the medial and lateral hinge centers are in alignment and the hinge arms are in midline of leg.
4. First secure strap above and below knee then secure the other straps. Remove as one unit. Open straps, open liner and remove as one unit.

SETTING RANGE OF MOTION STOPS ON HINGE

To open, flip security latch up (A). Rotate the two halves of the cap (B) to the side exposing the hinge plate. Lift pins out and position in desired settings. Make sure pins pass through holes in both top and bottom plates, and are completely seated. For proper functioning, the FLEXION and EXTENSION settings need to be identical on each side of the polycentric hinge plate, and identical for the medial and lateral hinges.
To close, rotate the two halves (B) to the center and “Click” closed. Push the security latch (A) down and “Click” closed.

The hinge arms can be carefully bent—foam liner and plastic paddle can be trimmed.

CLEANING

Foam liners and straps can be machine wash in 40°C gentle cycle and air dry. Wipe plastic and hinge with damp towel.

Contra indications—Fragile, non-intact skin.

Avoid placing near open flame, heater or in a closed car on a hot day.

Immediately contact your healthcare professional if you are experiencing skin irritations, pain, loss of sensations or if you have any questions or concerns.

Single patient use. To be used only when ordered by a physician and applied by qualified medical professional.

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